

Fracture Case Studies

Patient 1:

37 Year Old Male involved in a mountain bike accident. He sustained multiple comminuted fractures of mid shaft Tibia and Fibular.

Treatment Protocol

BodiHealth Device:

First Week: daily treatments for 5 days. There after treatments 3 times per week. All treatments were one ‘A’ cycle, followed by ‘5Z’

Results:

Had very little movement in the ankle which improved

to nearly full range of movement with little pain after first session. Experienced reduction of swelling and increased circulation. Soft tissue healing very evident over week one.

13 Months After the Accident:

Running 8 km without much pain afterwards.

15/6/04

Post Operage XR with tibial Rod2
17/12/04

Fracture after 5 months of standard therapy, fracture mal-union, minimal healing, advised by orthopaedic surgeon to have multiple bone grafts
31/01/05

Fracture after 7 weeks treatment with the Bodihealth devices

Patient 2

Mr SK of Adelaide, sustained a comminuted fracture of his talus.

Open Reduction Internal Fixation with screws.

3 weeks following operation still excruciating pain and debility.

He was told by the orthopaedic surgeon that he would require at least three months on painkillers and crutches before he could even begin to walk normally again.

Treatment Protocol

BodiHealth Device:

Three Weeks: Three treatments per week.

Results:

Mr SK was able to walk normally again pain free without crutches, drugs, or any swelling or inflammation in his ankle. "I've never seen such an amazing recovery from a fractured talus. The absence of swelling is particularly remarkable." Dr. Z. Baran.

Patient 3

60 year old female fractured left fifth toe 27 October 2004. She had not recovered as expected. Walking had become increasingly difficult as she had severe pain with weight bearing requiring crutches for 6 months.

CT scan April 2005 – possible navicular stress fracture, osteopaenia, some soft tissue atrophy.

Treatment Protocol:

BodiHealth Device:

3 treatments: Wrapped both feet.

"A"; one cycle "3"; 2 1/2 hours

Results:

First Treatment: Decrease in Pain by 50%.

On pain scale was 10/10 now 5/10

Second Treatment: Excellent result pain decreased by 95%. Walked in today without crutches.

Third Treatment: Patient was not using crutches

and had NO pain.